FENTANYL DRUG CHECKING STRIP



Put your drugs (a few grains of salt amount) into a shot glass amount of tap water





Dip the strip into the cup for at least 10 seconds, no deeper than the wavy line





Wait for red stripe to appear

1 LINE: YES fentanyl



2 LINES: NO fentanyl

STIMULANTS CAN CAUSE FALSE POSITIVES.
ADD MORE WATER AND REPEAT.

Strips cannot detect all types of fentanyl, such as carfentanyl

REGARDLESS OF RESULTS, ALWAYS STAY SAFE NEVER USE ALONE * CARRY NALOXONE *